




Directions for use

Landing Med cloud diagnostic platform

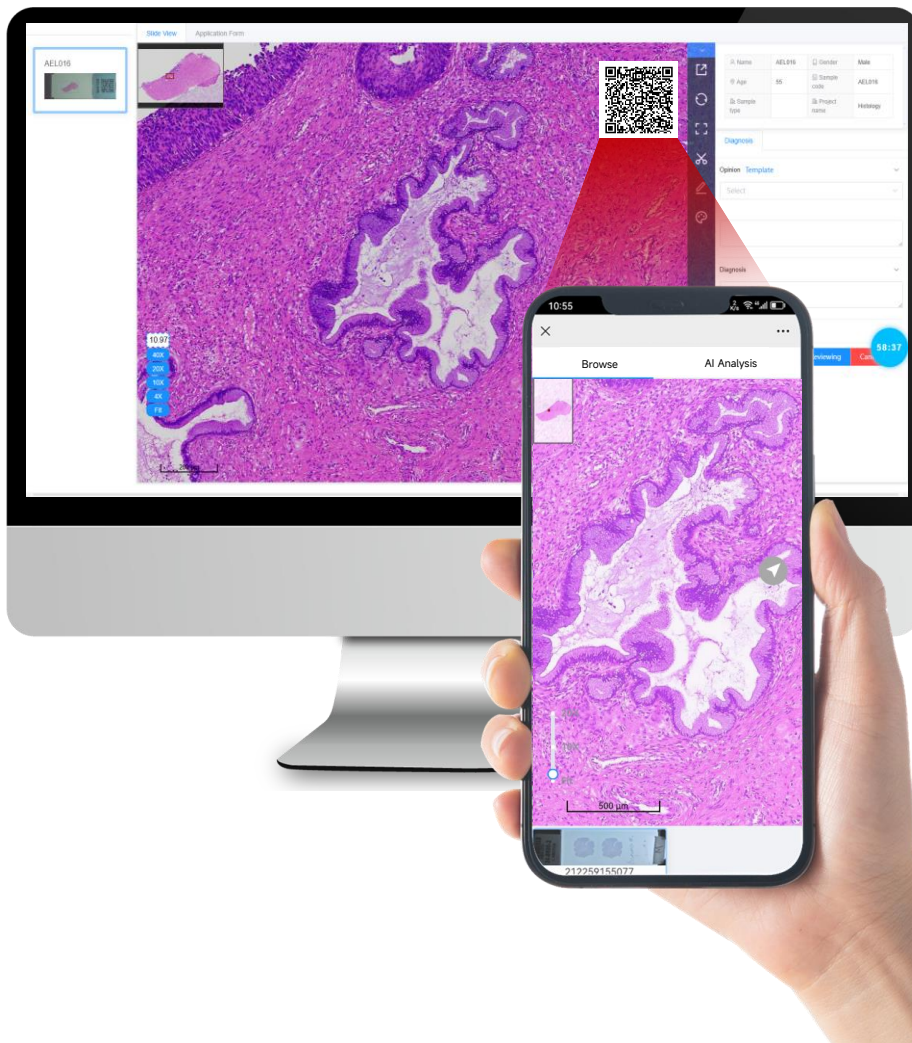


Website: www.landingmed.com
Email: hello@landing-med.com



1.Log in to the cloud platform

URL : c3global.landingmed.cn

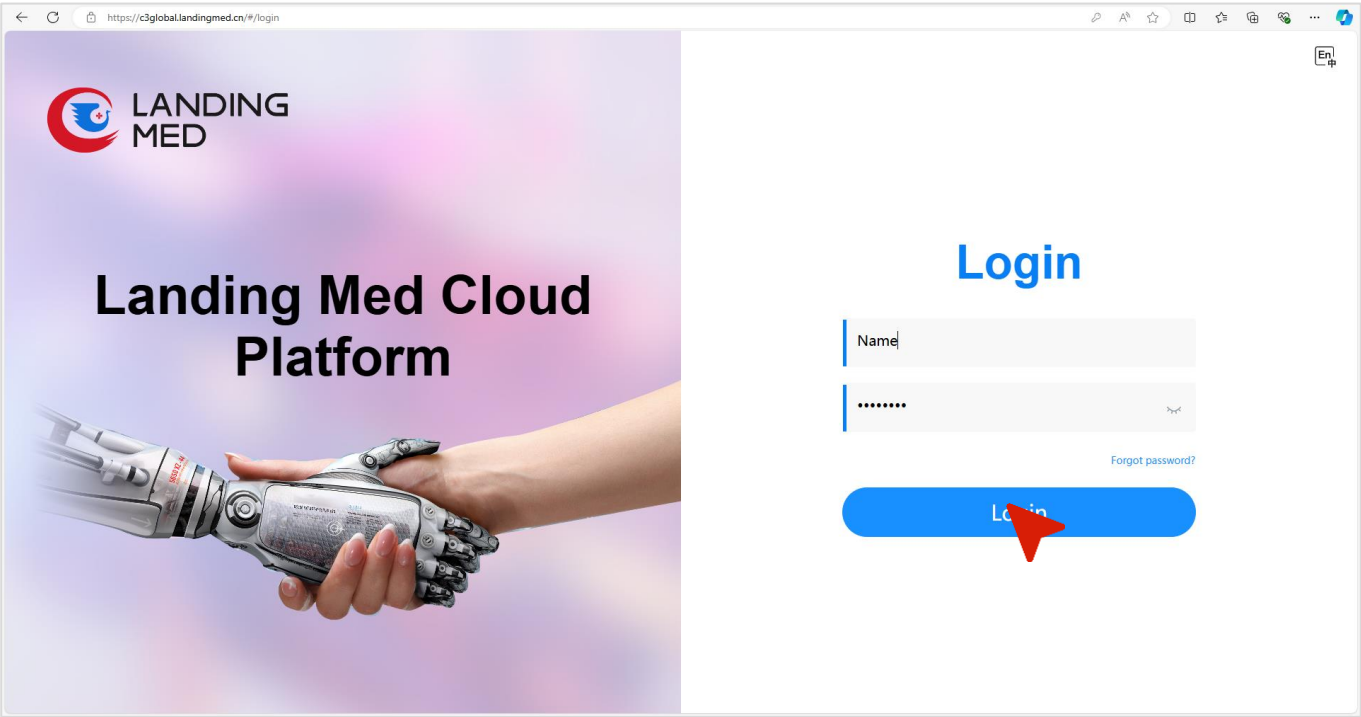

Enter your account password and click “ Login ”

2. Query the sample number

Click on the left “ Data management ”

Home

Information entry

Primary diagnosis

Data management

Home / Scan management / Data management

Home | Information entry | **Data management**

Data management

Refresh

Sequence	Sample number	Sample code	Sample ID	Device name	Status	Scan hospital	Registration status	Cloud path	Creation time
1	241138094451	241138094451	56987710-6237-11ef-a519-009027e9330a	LD Patho 320A 342202 30036	AI processing completed Preview Share	PER	<div>✓</div>	SDM_DATA/2024-08-25/LD Patho 320A 34220230036/56987710-6237-11ef-a519-009027e9330a	2024-08-25 00:41:02
2	232326207926	232326207926	f2cbb5dc-61d1-11ef-9daa-009027e9330a	LD Patho 320A 342202 30036	Splicing completed Preview Share	PER	<div>✗</div>	SDM_DATA/2024-08-24/LD Patho 320A 34220230036/f2cbb5dc-61d1-11ef-9daa-009027e9330a	2024-08-24 22:13:30
3	232326207427	232326207427	27bc3d55-61d1-11ef-9db2-009027e9330a	LD Patho 320A 342202 30036	Splicing completed Preview Share	PER	<div>✗</div>	SDM_DATA/2024-08-24/LD Patho 320A 34220230036/27bc3d55-61d1-11ef-9db2-009027e9330a	2024-08-24 12:37:05
4	232303080157	232303080157	ecbd7957-61cd-11ef-8dd0-009027e9330a	LD Patho 320A 342202 30036	Splicing completed Preview Share	PER	<div>✗</div>	SDM_DATA/2024-08-24/LD Patho 320A 34220230036/ecbd7957-61cd-11ef-8db0-009027e9330a	2024-08-24 12:10:15
			7b51b4bd-61a8-11ef-8f55-009027e9330a	LD Patho 320A 342202 30036	Splicing completed			SDM_DATA/2024-08-24/LD Patho 320A 34220230036/7b51b4bd-61a8-11ef-8f55-009027e9330a	

10/page < > Go to 1

2

1

2. Copy unregistered samples


1.Check the status of your sample

✗ No Registered

✓ Registered ✗ No Registered

3. Sample registration

Click on the left “Information entry”

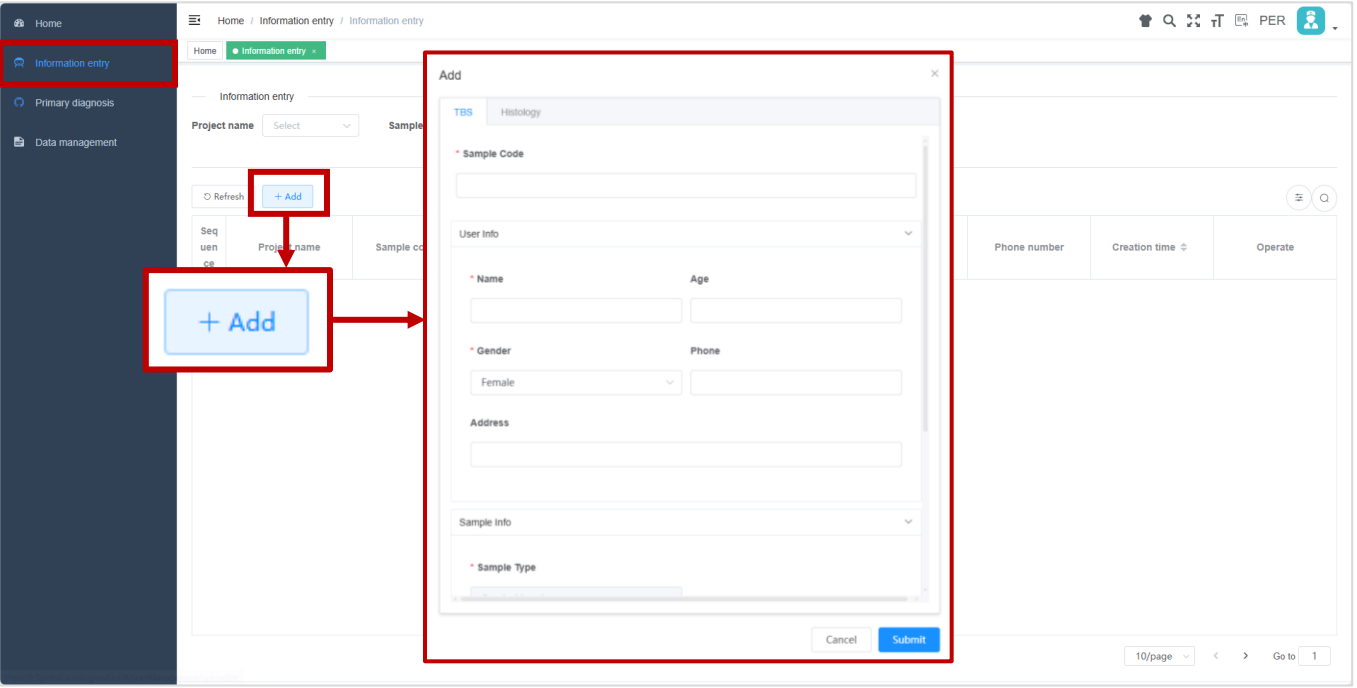


Select a sample type

“*” Required, but can insert test name for testing

Paste the copied sample number


Select whether you want AI or not

Fill in and click “Submit”

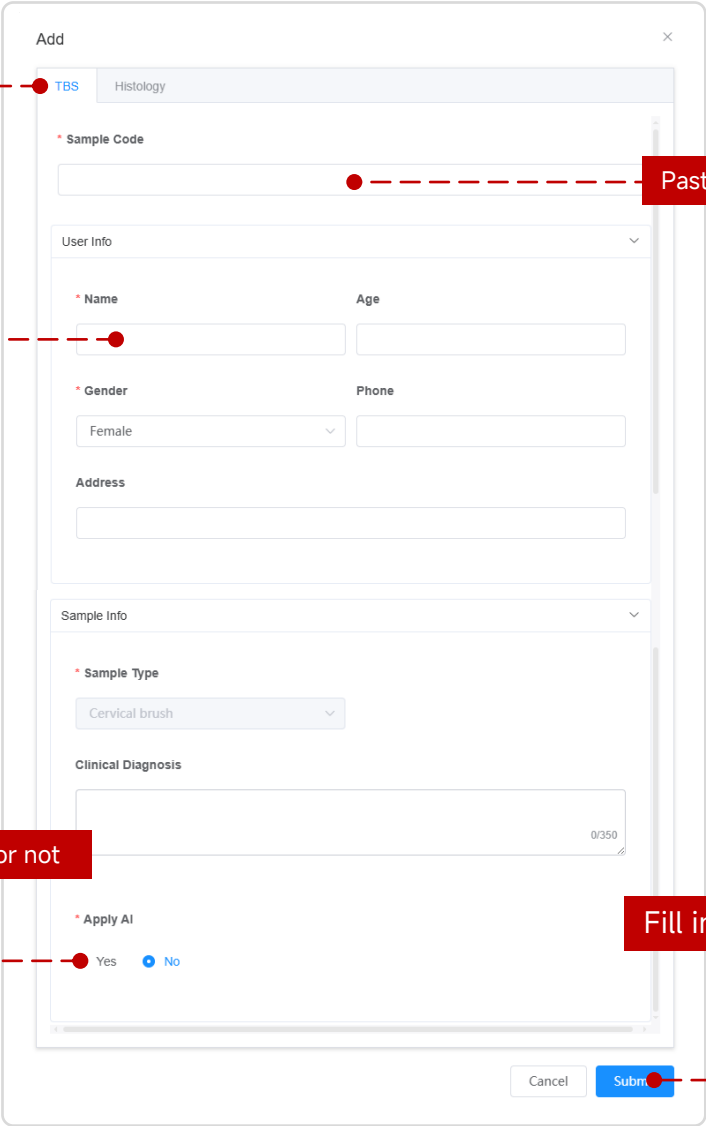

4. Diagnose and share samples

Click on the left “Primary diagnosis”

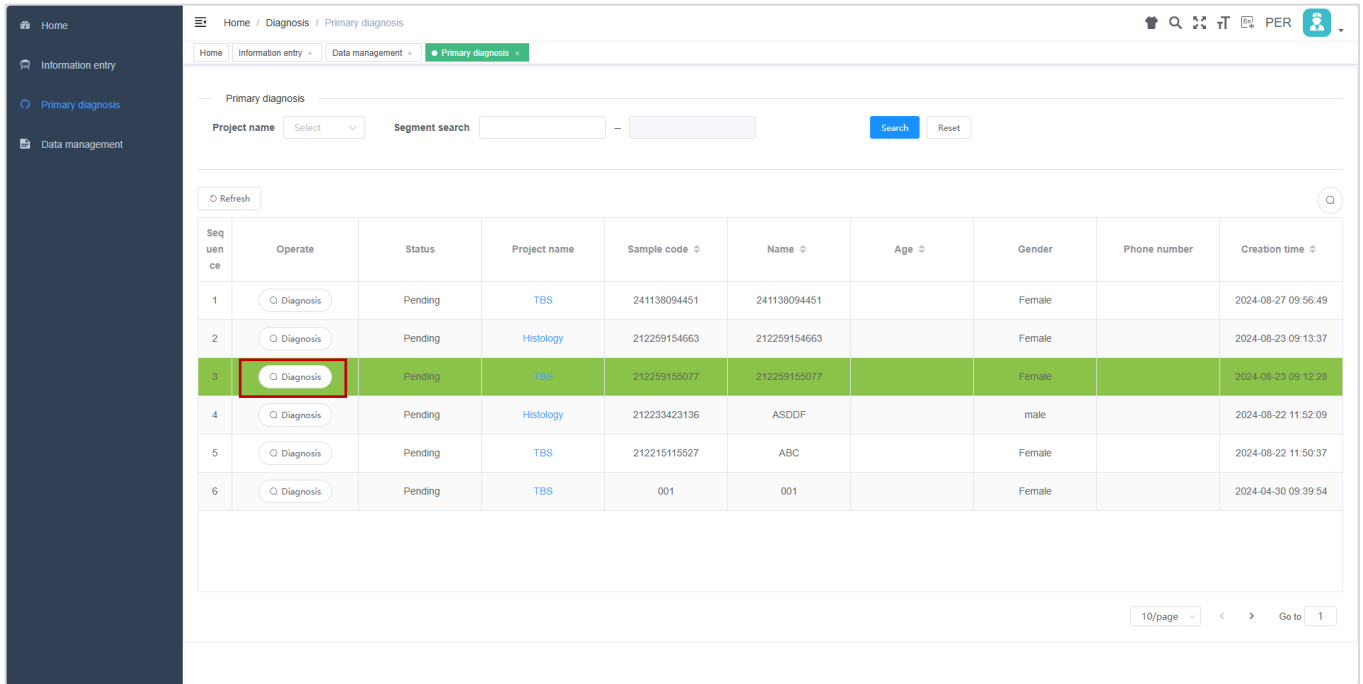


The interface shows a sidebar with navigation options: Home, Information entry, Primary diagnosis (selected), and Data management. The main area is titled 'Primary diagnosis' and includes a search bar with 'Project name' and 'Segment search' dropdowns, and 'Search' and 'Reset' buttons. Below the search bar is a table with the following columns: Sequence, Operate, Status, Project name, Sample code, Name, Age, Gender, Phone number, and Creation time.

Sequence	Operate	Status	Project name	Sample code	Name	Age	Gender	Phone number	Creation time
1	<input type="button" value="Diagnosis"/>	Pending	TBS	241138094451	241138094451		Female		2024-08-27 09:56:49
2	<input type="button" value="Diagnosis"/>	Pending	Histology	212259154663	212259154663		Female		2024-08-23 09:13:37
3	<input type="button" value="Diagnosis"/>	Pending	TBS	212259155077	212259155077		Female		2024-08-23 09:12:28
4	<input type="button" value="Diagnosis"/>	Pending	Histology	212233423136	ASDDF		male		2024-08-22 11:52:09
5	<input type="button" value="Diagnosis"/>	Pending	TBS	212215115527	ABC		Female		2024-08-22 11:50:37
6	<input type="button" value="Diagnosis"/>	Pending	TBS	001	001		Female		2024-04-30 09:39:54

At the bottom right, there is a pagination control showing '10/page', navigation arrows, and 'Go to 1'.


1. Click " " to view the sample to be diagnosed

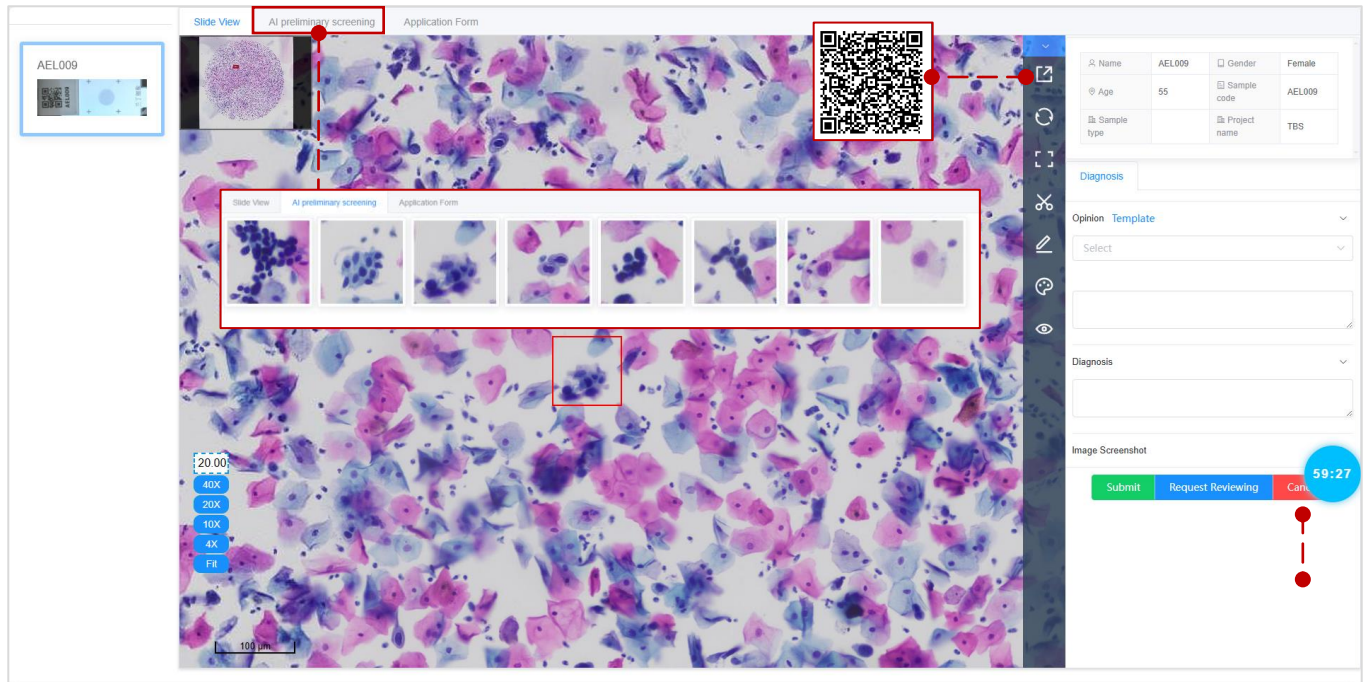

The interface shows a histology slide with a red box highlighting a specific area. A red dashed line connects this area to a red box labeled 'AI preliminary screening' at the top. Below the main slide, there is a row of six small thumbnail images, with the first one highlighted by a red box. To the right of the main slide is a QR code. On the far right, there is a sidebar with patient information: Name (AEL009), Age (55), Gender (Female), Sample code (AEL009), Sample type, and Project name (TBS). Below this is a 'Diagnosis' section with a 'Template' dropdown and a 'Select' button. At the bottom right, there is a 'Image Screenshot' section with 'Submit', 'Request Reviewing', and 'Cancel' buttons, and a clock showing '59:27'.

2.Scroll to zoom in and out

3. If there is an AI, look at the AI data

4. Click "  " to share and browse the QR code


5. Share is complete, " " exit the sample

Landing Med cloud: Precautions

PC browsing recommended for reviewing "Primary Diagnosis" page. Mobile browsing is not supported for functions within "Primary Diagnosis" except viewing the Whole Slide Image and AI Preliminary Screening cells




1. The screen content is incomplete, or only one sample can be displayed

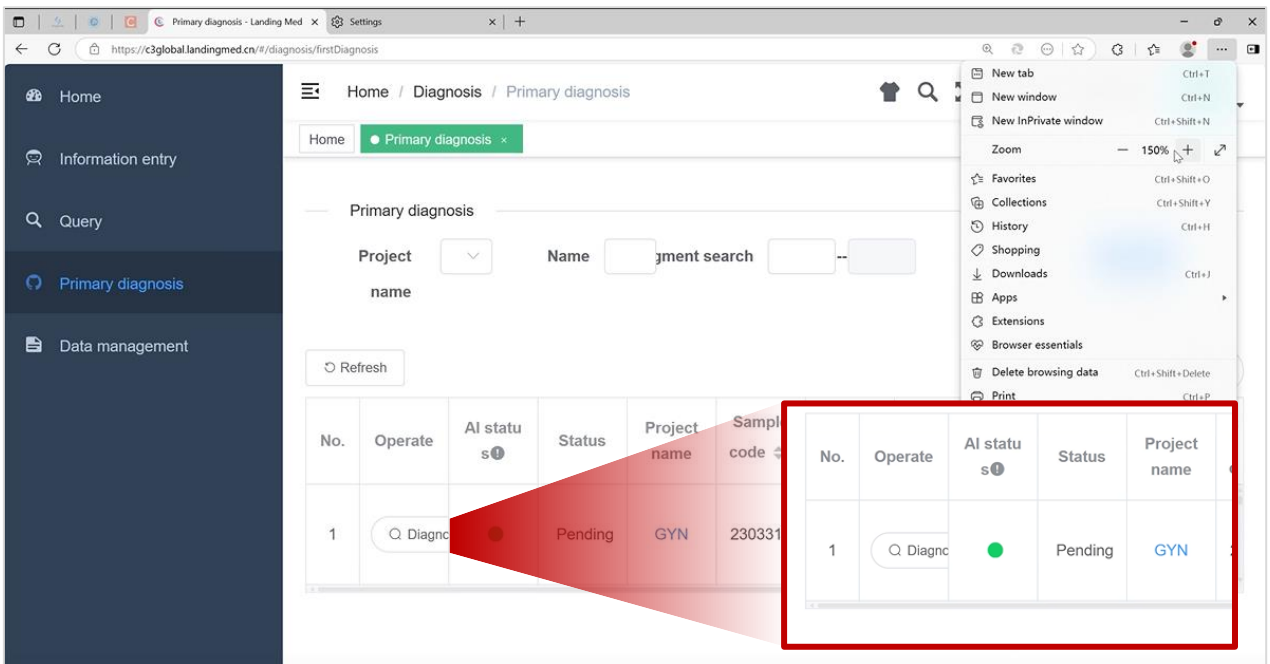

The screenshot shows the "Primary diagnosis" page in a web browser. The page has a sidebar with navigation options: Home, Information entry, Query, Primary diagnosis (selected), and Data management. The main content area shows a search bar and a table with columns: No., Operate, AI status, Status, Project name, and Sample code. Only one row is visible, with a red arrow pointing to it.

No.	Operate	AI status	Status	Project name	Sample code
1	Q Diagn	●	Pending	GYN	230331

2. Click on the upper right corner of the browser to adjust the display scale in the settings
100% is recommended, > 100% content will be less, < 100% will show more content

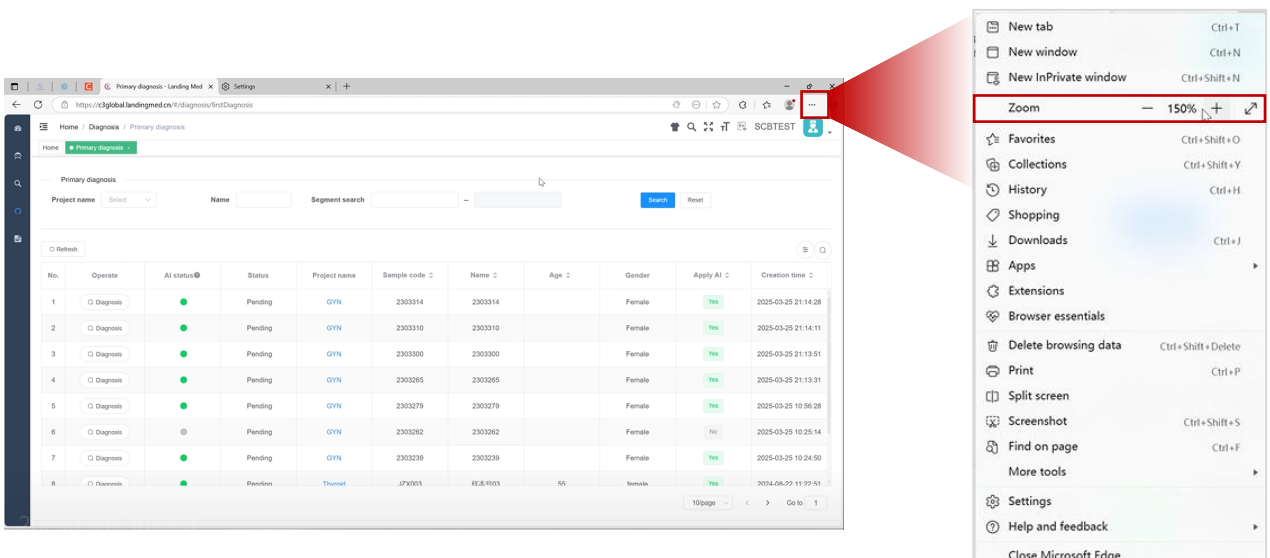

The screenshot shows the "Primary diagnosis" page in a web browser. The page has a sidebar with navigation options: Home, Information entry, Query, Primary diagnosis (selected), and Data management. The main content area shows a search bar and a table with columns: No., Operate, AI status, Status, Project name, Sample code, Name, Age, Gender, Apply AI, and Creation time. Multiple rows are visible. A red arrow points to the browser's zoom settings in the top right corner.

No.	Operate	AI status	Status	Project name	Sample code	Name	Age	Gender	Apply AI	Creation time
1	Q Diagn	●	Pending	GYN	2303314	2303314		Female	Yes	2025-03-25 21:14:28
2	Q Diagn	●	Pending	GYN	2303310	2303310		Female	Yes	2025-03-25 21:14:11
3	Q Diagn	●	Pending	GYN	2303300	2303300		Female	Yes	2025-03-25 21:13:51
4	Q Diagn	●	Pending	GYN	2303285	2303285		Female	Yes	2025-03-25 21:13:31
5	Q Diagn	●	Pending	GYN	2303279	2303279		Female	Yes	2025-03-25 10:56:28
6	Q Diagn	●	Pending	GYN	2303262	2303262		Female	No	2025-03-25 10:25:14
7	Q Diagn	●	Pending	GYN	2303239	2303239		Female	Yes	2025-03-25 10:24:50
8	Q Diagn	●	Pending	Throat	JPX003	88J81003	65	Female	Yes	2024-08-29 11:22:41

Cross-platform sharing and browsing

It's as simple as that



Website: www.landingmed.com
Email: hello@landing-med.com

